

## Disinfection Bath

---

- » Resistant to heat
- » Easily cleaning
- » Easily mounting



# Disinfection Bath

## Application Area

Disinfection Bath is presented to be used in disinfection and cleaning of all kind of medical and surgical instruments including endoscopes.

## Features

It contains a filter and a cover for doing the disinfection and the cleaning easily and for preventing the contamination of the staff. It can be used with either the concentrated or the ready-to-use products. It contains a dosage table for the using of the disinfectants properly and a separate table for the recording of the procedure. The bath is resistant to the cleaning agents and can be cleansed easily. It is made of polyethylene and it has antibacterial characteristics. The disinfection bath can be used with hot water since it is resistant to heat.

## Usage

When using of the concentrated product, put the water and the product into the disinfection bath checking the dosage table. When using of the ready-to-use, put the product into the disinfection bath without diluting. Place the equipments into the bath and record the procedure onto the cover. After waiting of effect duration open the cover and take the equipments out of the bath by holding the handles.

## Precautions

Do not keep the cover open during the disinfection procedure!  
Do not expose to the heat or fire!



### ACTO® Disinfection Bath

Packaging

Single Unit	Size	Packaging Unit	Article-No.
3 L	300x190x115	1	10.0004.3



### ACTO® Disinfection Bath

Packaging

Single Unit	Size	Packaging Unit	Article-No.
10 L	390x290x170	1	10.2192.10



### ACTO® Disinfection Bath

Packaging

Single Unit	Size	Packaging Unit	Article-No.
30 L	700x300x180	1	10.2202.30